



sutherland medical

Suportx Neo Easy Peel Belt

FIRM SUPPORT

SIZE	MEASUREMENT	COLOUR	DEPTH OF BELT		
			15CM	20CM	26CM
SMALL	75-90CM / 30-36ins	Black	SPX2251DB	SPX2261DB	SPX2271DB
MEDIUM	90-105CM / 36-42ins	Black	SPX2252DB	SPX2262DB	SPX2272DB
LARGE	105-120CM / 42-48ins	Black	SPX2253DB	SPX2263DB	SPX2273DB
X LARGE	120-135CM / 48-54ins	Black	SPX2254DB	SPX2264DB	SPX2274DB
XX LARGE	135-148CM / 54-59ins	Black	SPX2255DB	SPX2265DB	SPX2275DB
SAS ALLOWANCE: 4 GARMENTS ALLOWED PER ANNUM (PACK SIZE: 1)					
SIZE	MEASUREMENT	COLOUR	DEPTH OF BELT		
			15CM	20CM	26CM
SMALL	75-90CM / 30-36ins	Neutral	SPX2251DS	SPX2261DS	SPX2271DS
MEDIUM	90-105CM / 36-42ins	Neutral	SPX2252DS	SPX2262DS	SPX2272DS
LARGE	105-120CM / 42-48ins	Neutral	SPX2253DS	SPX2263DS	SPX2273DS
X LARGE	120-135CM / 48-54ins	Neutral	SPX2254DS	SPX2264DS	SPX2274DS
XX LARGE	135-148CM / 54-59ins	Neutral	SPX2255DS	SPX2265DS	SPX2275DS
SAS ALLOWANCE: 4 GARMENTS ALLOWED PER ANNUM (PACK SIZE: 1)					

Suportx Stoma Shield


Suportx Stoma Shield	
SUPORTX STOMA SHIELD	
CODE	SSS900W
SAS CODE: 80348W	SAS ALLOWANCE: 1 STOMA SHIELD ALLOWED PER ANNUM (PACK SIZE: 1)

Support Belt Measuring Instructions


Measuring the width of a support belt

Measure both lying down and standing up. The smaller measurement of the two should be used for required level of compression.

1. Place the tape around the widest part of the body where the belt is to be worn/support is needed. This is normally over the hernia/stoma pouch if relevant. Keep the tape measure fairly tight.

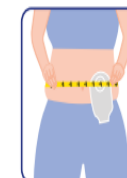

Measure standing up

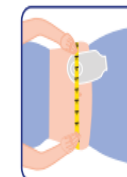

Measure laying down

Measuring the depth of support belt

1. Establish where the top of the belt needs to be positioned
 - a. This will normally be above the top of the pouch by at least 3cm or above the hernia by at least 3cm whichever is the highest.
2. Once the top position has been established, put the tape in a straight line down the body to where the customer would like the belt to finish, this will need to fully cover the hernia/abdomen area without digging into groin/legs.



Measure standing up



Measure laying down

Zoe Bailey | Sutherland Medical

Ph: 1300 664 027 – Mob: 0408 376 927 Email: zoe@sutherlandmedical.com.au

Address: Sutherland Medical, Unit 2, 195 Chesterville Road Moorabbin, VIC 3189